



# Summary of Instructions and Warnings for Using the **PAL – Portable Aquatic Lift**

Read and Follow all User Instructions listed in the  
Owner's Manual before use  
Read and Follow all Caution, Warning labels located  
on the lift before use

## Warnings and Safety Summary



**DANGER – FAILURE TO FOLLOW THESE WARNINGS,  
INSTRUCTIONS AND THE OWNER'S MANUAL MAY  
RESULT IN SERIOUS INJURY OR DEATH**

### **WARNING**


- **ADULT USE ONLY** – Use by children with adult supervision only. Only persons healthy enough for water activities should use the PAL. Users should consult with their physician to determine if water activities are appropriate for the User.
- Seat belt is to be used at all times during user transfer.
- Lift must be positioned on firm level ground not to exceed a slope of 2°.
- **NO JUMPING/DIVING/CLIMBING** – Do not jump/dive from or climb on lift.
- **TIP OVER HAZARD** – Stabilizing arms must be fully extended before each use and must be used at all times when a user is seated. The lift may not be used without the stabilizing arms fully extended. Never attempt to retract the stabilizer arms when the lift is in use. Do not reposition the lift when lift is in use.
- **UNINTENDED MOVEMENT/TIP OVER HAZARD** – Rear wheels must be locked at all times while the lift is in use. Ensure proper ballast weights are installed in base before each use.
- **TIP OVER HAZARD** – Do not use the lift to transport a user from place to place.
- **TIP OVER HAZARD** – Limit for use is ONE PERSON weighing less than the maximum weight limit of - 300lbs (136kg). Use only for transfer of user not as a hoist or crane.
- **PINCH/CRUSH HAZARD** – Always keep hands and fingers clear of lift arms during use.
- The battery must be completely removed from lift and located away from pool deck for charging.

### **CAUTION**

- **BATTERY CHARGE LEVEL** – Check Battery Level before each use. Battery must show GREEN >50% charge level before use. When YELLOW <50% charge level is shown the battery must be fully charged before use. When RED <25% charge level is shown the lift must not be used and the battery must be fully charged before use.
- **DAMAGE MAY RESULT**– Do not use the seat or seat frame to move or position the PAL.
- **DAMAGE MAY RESULT**- Do not allow excessive moisture to collect in the battery case, it can affect battery and lift performance and could lead to battery failure and/or the lift failing to operate.


## POSITIONING THE PAL

- Do maintain a firm grip on the handle - use care when positioning the PAL, it is heavy (855 lb/401 kg).
- Do position the PAL on firm level ground not to exceed a slope of 2° and in an area that allows plenty of room for transferring to and from a wheelchair.
- Locate the PAL so that the front of the lift is facing the edge of the pool within a minimum distance of 2”(5cm) to a maximum distance of 24”(61cm) set back from the edge of the pool.
- Do fully extend both stabilizing arms.
- Do lock both rear wheels.

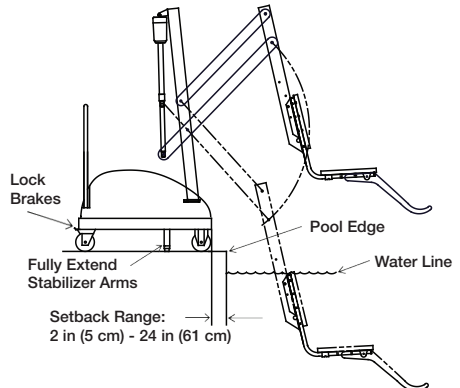

## TRANSFERRING

- Do transfer onto seat, ensuring user's weight is centered on seat.
- Do fasten seat belt or optional stability vest.
- Do rotate seat to 12:00 position, over water.
- Do lower seat into pool.
- Do unfasten seat belt or optional stability vest – transfer into water.
- Do when finished, return to seat, ensuring user's weight is centered on seat.
- Do fasten seat belt or optional stability vest.
- Do rotate seat to original transfer position of 12:00.
- Do raise/lower seat to proper transfer height.
- Do unfasten seat belt or optional stability vest – transfer off seat.

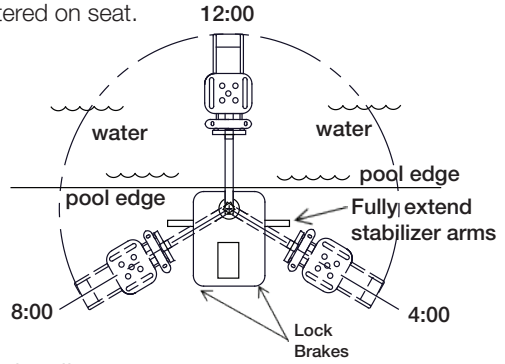

## MAINTENANCE and CLEANSING

- Do keep all electronic components clean and dry.
- Do keep the Console/Battery cover installed at all times.
- Do not allow disinfection solution to have direct contact with the skin and eyes.
- Do refer to Owner's manual for weekly / monthly lift maintenance and cleansing.

Perform the following schedule to ensure proper operation before each use:

Maintenance Performed Before Each Use	Daily	Weekly	Monthly
Check battery level before each use / Charge battery daily	✓		
Wipe Control Box and battery connection with a clean dry rag	✓		
Examine lift for any damage, loose or missing hardware	✓		
Test all control buttons (hand and control box) for normal operation	✓		
Spray gear assembly with a heavy-duty rust inhibitor/lubricant such as LPS 3 - Heavy-Duty Inhibitor		✓	
Make sure all cable connections are properly secured			✓
Inspect lift frame, mast, support arm and seat assembly for rust			✓
Inspect weight stack for rust/treat with touch up paint			✓
<b>Cleansing Performed After Each Use</b>			
Rinse seat and seatbelt/stability vest with fresh water between each use - Cleanse seat and seatbelt/stability vest with a disinfectant solution of 1:100 dilution of household bleach to fresh water and then rinse with fresh water and dry entire lift daily. In the event of a contamination incident such patient/user excreta - cleanse seat and seatbelt/stability vest immediately with the disinfectant solution*	✓		
Cleanse all battery connections with a nylon scouring pad			✓
Cleanse all metallic surfaces with a cleaner wax to maintain the finish of the lift			✓

\* When using the disinfection solution, avoid direct contact with the skin and eyes. In the event of a contamination incident, immerse the seat belt or stability vest in the disinfection solution for 10 min. and then rinse thoroughly with fresh water.



S.R. Smith, LLC  
 PO Box 400  
 1017 SW Berg Parkway  
 Canby, Oregon 97013 USA  
 Phone: 503-266-2231  
 Fax: 503-266-4334  
 www.srsmith.com  
 Made in USA



QNET B.V.  
 Hommerterweg 286  
 6436 AM Amstenrade  
 The Netherlands  
 qnet@ce-mark.com  
 qnet@ce-authorizedrepresentative.eu

