

Beach Access Chairs



F-013BAC

F-013BAC (4 Large Wheels)
F-014BAC (Rear Articulated Wheels)

250 LB. [113 kg] MAX WEIGHT CAPACITY

MANDATORY – LEAVE THIS MANUAL WITH OWNER



Read & follow all instructions
Safety can only be ensured if the beach chair is assembled according to these instructions



NEVER permit children to play on or around the beach chair. Do not allow children to use the chair without adult supervision



DO NOT use beach chair without checking tires for proper inflation pressure. Chair may be unstable or unsafe if the tire pressure is too high or too low



NEVER operate the beach chair in water
This product is not a floatation device



F-014BAC

9889 Garrymore Ln
Missoula, MT 59808
888-687-3552 | +1-406-549-0769
aquacreek.com



SAVE a copy of these instructions

Parts List



ITEM #	QTY	DESCRIPTION
1	1	BEACH ACCESS CHAIR
2	1	UMBRELLA
3	1	UMBRELLA POLE

READ CAREFULLY

Check entire box and inside all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical all parts be carefully inspected prior to assembly to ensure no damage occurred in transit and a damaged part is not used. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper assembly cannot be overstressed, as an improper assembly voids Aqua Creek's warranty and may affect the safety of the user.

Assembly Instructions

The Beach Access Chair comes fully assembled in a crate. All the user needs to do is to check the tire inflation, assemble the umbrella and install it on the chair. The instructions below explain how this is done.

Checking Tire Pressure

The tire pressure should be checked before using the Beach Access Chair and periodically afterwards to ensure that the chair is safe and stable. To check the tire pressure, remove the tire valve cap from the tire and check the tire pressure using a standard tire pressure gauge (not included). The tire pressure range is listed on the side of each tire, but is typically 2 to 4 PSI. If the tire pressure is low, use an air pump to increase the pressure to the rated range. Use the pressure gauge to check the pressure after inflation.

IMPORTANT: Be sure to replace the tire valve cap after checking and adjusting the tire pressure.




Removing Tire Valve

Umbrella Assembly & Installation

The umbrella comes in a separate plastic bag, and needs to be assembled before use. To assemble the umbrella, insert the umbrella (top section) into the umbrella pole as shown. Adjust the umbrella to the desired height and lock it in place using the locking handle as shown.



Now the umbrella can be installed into the umbrella hole as shown below

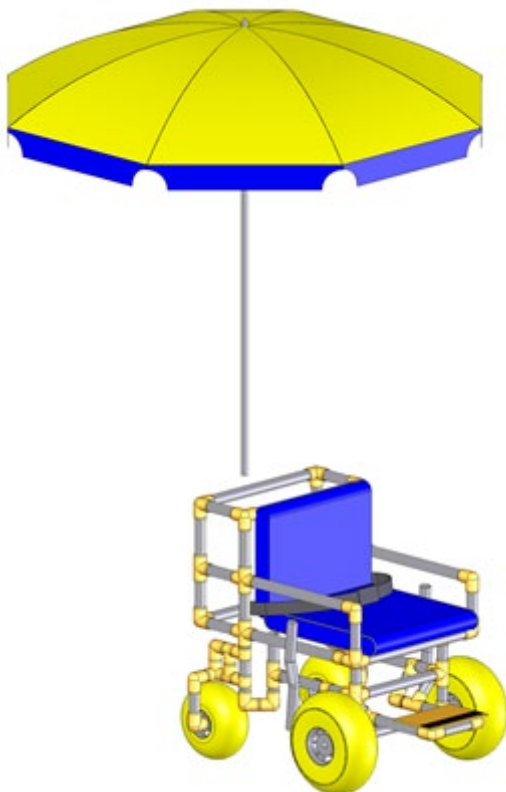

Operating Instructions

The Beach Access Chair operates in an assisted mode; one person rides in the chair while another pushes it from behind. **ALWAYS USE THE SAFETY BELT WHEN RIDING IN THE BEACH ACCESS CHAIR.**

Using the Parking Brake

When the Beach Access Chair is stopped the parking brake should always be engaged. To engage the parking brake, rotate the brake lever back and insert the stainless steel nut into the hole in the side tube as shown below



Engaging the Parking Brake




Parking Brake Engaged

To release the brake, pull the brake lever nut out of the hole (sideways) and rotate the brake handle forward as shown below






Parking Brake Disengaged

Disassembly F-013BAC

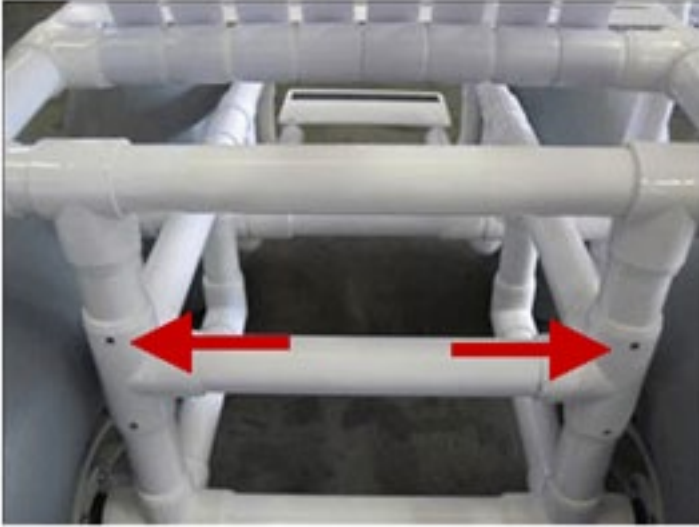

- 1) Take 1 pin out to remove each of the 4 wheels. Take wheels off axle.
- 2) To disassemble further, take the 4 pins out (2 on front side, 2 on back side). Remove the top set of pins located below the seat area.
- 3) After these 4 pins are removed, the top section (seat part of the chair) will detach from the bottom section (legs of chair and bottom base). It comes apart at the top of the straight legs of chair. Use a rubber mallet to separate the sections.

Disassembly F-014BAC

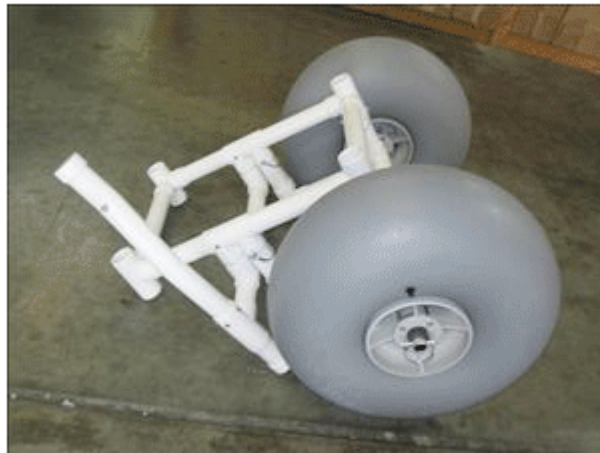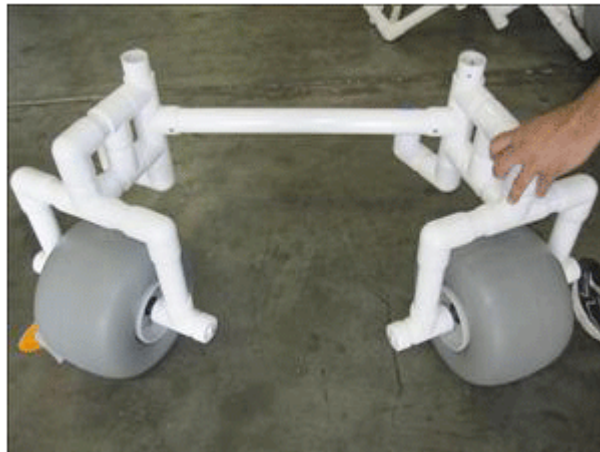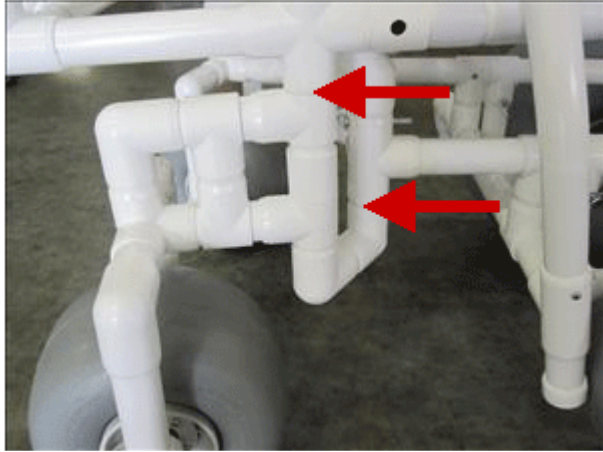

1) Remove the swivel wheels in back. (This contains the wheel itself, along with the PVC pipe housing that allows it to swivel). It is all one piece. Take two D ring pins out for each swivel. Then tap on the top of the elbow until swivel part disconnects from chair. Do the same for both swivels

2) Take the two D ring pins off to remove the 2 large wheels.

3) To remove further, take the top set of pins (2 front & 2 back) under the seat area out. Once these 4 pins are removed the top part of the frame (seat part of chair) will separate from the bottom section. Bottom section consists of everything below the seat frame. Use a rubber mallet to separate the sections.

**AQUA CREEK PRODUCTS, LLC LIMITED ONE (1) YEAR WARRANTY:
BEACH ACCESS CHAIRS
(ITEM #'s F-013BAC, F-014BAC, F-015RBAC, F-016BS)**

Aqua Creek Products, LLC (a.k.a. Aqua Creek) warrants to the original end user purchaser that products manufactured by Aqua Creek, when properly installed in accordance with assembly and installation instructions and properly used and maintained, shall be free from defects in material and workmanship for a period of one (1) year from the date of original purchase, provided that Aqua Creek receives prompt notice in writing of any defect or failure and satisfactory proof thereof, with the following exceptions:

Exceptions:

- This warranty specifically excludes reimbursement for labor to remove, repair, or install the product and any freight charges.
- This warranty does not cover any damages due to accident, misuse, abuse, negligence or failure to properly maintain any products, or normal wear and tear from day to day operations.
- In the event that any products are altered, repaired, or improperly installed by anyone without the prior written approval by Aqua Creek, all warranties are void.

IMPORTANT: User is not to exceed the rated weight capacity of this product. It is the responsibility of the product owner to verify the weight of the patron for warranty claim purposes and to ensure that the product is not overloaded.

To initiate a warranty claim, the owner of an Aqua Creek product must provide the place of purchase, in writing, with a full description of the product, its serial number, the dates of purchase and installation, and the exact nature of the defect. Within thirty (30) days after receipt of a written warranty claim by Aqua Creek, and barring any unforeseen delays, the place of purchase will be notified of Aqua Creek's decision regarding the claim. All warranty calculations are from date of original purchase at manufacturer's suggested retail price. Non-payment for product to Aqua Creek may void warranty.

If requested by Aqua Creek, any defective product must be returned, freight prepaid, to Aqua Creek's designated factory location or duly appointed distributor for inspection and/or repair. Aqua Creek will, at its option, repair or replace the failed or defective item, and deliver the repaired product or replacement to the seller of the product, freight prepaid to the destination provided for in the original order. Products returned to Aqua Creek for which Aqua Creek provides replacement under this limited warranty shall become the property of Aqua Creek.

A new warranty period shall NOT be established for the repaired or replaced products. Such products shall remain under warranty only for the remainder of the original warranty period on the original products purchased.

This written limited warranty constitutes the final, complete and exclusive statement of warranty terms. No person or organization is authorized to make any other specific or implied warranties or representations on behalf of Aqua Creek.

THE WARRANTIES SET FORTH HEREIN ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED, INCLUDING WITHOUT LIMITATION ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE.

THE SOLE AND EXCLUSIVE REMEDIES FOR BREACH OF ANY AND ALL WARRANTIES WITH RESPECT TO THE PRODUCTS SHALL BE LIMITED TO REPAIR OR REPLACEMENT AT AQUA CREEK'S DESIGNATED FACTORY LOCATION, OR DULY APPOINTED DISTRIBUTOR, OR IN PLACE AT AQUA CREEK'S OPTION. IN NO EVENT SHALL AQUA CREEK'S LIABILITY EXCEED THE ENTIRE AMOUNT PAID TO AQUA CREEK BY THE ORIGINAL PURCHASER FOR THE FAILED OR DEFECTIVE PRODUCT.

IN NO EVENT SHALL AQUA CREEK PRODUCTS, LLC BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL, INDIRECT, PUNITIVE OR EXEMPLARY DAMAGES OR LOST PROFITS FROM ANY BREACH OF THIS LIMITED WARRANTY OR OTHERWISE.

Aqua Creek Products, LLC
9889 Garrymore Lane
Missoula, MT 59808
Toll Free: (888) 687-3552
Local/Intnl: (406) 549-0769
www.aquacreek.com