
Lift Remote Programming

To program a remote to you receiver use the following procedure.

1. Place battery on lift with switch in the off position. (For additional safety ensure that only the lift being programmed has a battery installed. After programming is completed all lifts may have a battery installed.)
2. Press and hold the unlock button on the remote for 5 seconds.
3. Press and hold the up button on the remote while switching the battery to the on position.
4. The lift should begin to move up if not repeat steps 1-3 until the lift moves up when the battery is switched on.

To remove a programmed remote

1. Place battery on lift with switch in the off position. (For additional safety ensure that only the lift being programmed has a battery installed. After programming is completed all lifts may have a battery installed.)
2. Press and hold the unlock button on the remote for 5 seconds.
3. Press and hold the up button on the remote while switching the battery to the on position.
4. The lift should begin to move up if not repeat steps 1-3 until the lift moves up when the battery is switched on.
5. Repeat steps 1-4 for a total of five times.

If you have any difficulties please contact Spectrum customer service at 406-532-6322